



- NOTES(技术要求):
- 1. 折叠后尺寸: 75*100mm
 - 2. 印刷颜色: 彩色印刷
 - 3. 材质: 80 g哑粉纸
 - 4. 表面处理: 无
 - 5. 装订工艺: 折叠
 - 6. 公差: 长X宽: +/-0.5mm

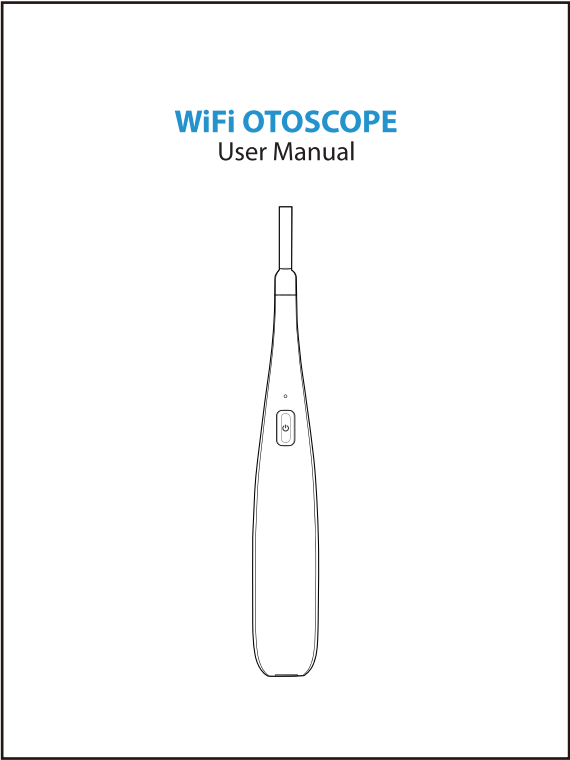

成品图

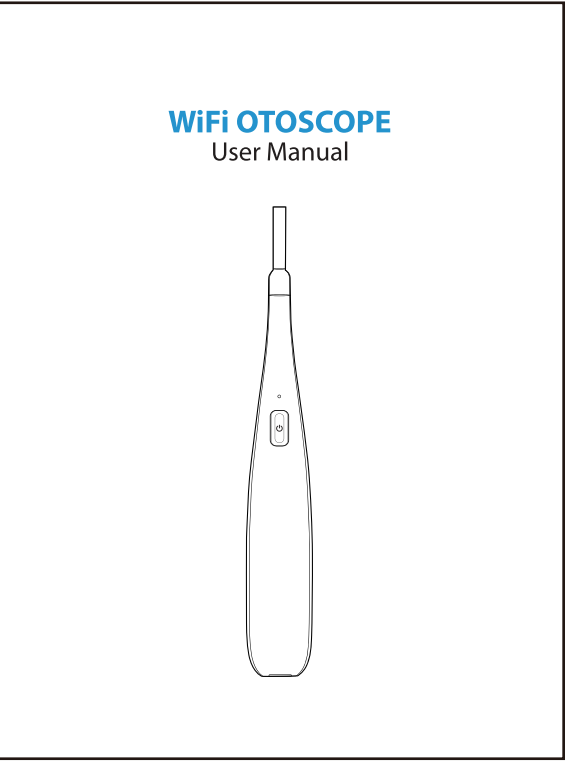

General Information

Manual:

This operation manual allows you to work safely with the Wi-Fi Endoscope camera. Please keep this manual for your Information.
Liability is void for any damage caused by not following this manual.

Maintenance:

To clean this device, use a damp cloth or mild, water based cleaner.
Fully charge the battery before you leave this device idle for long time to prevent damage to the battery.

Storage and transport:

Improper transport can harm the device. Always transport this device in the provided box in order to prevent damage.

Information on disposal:

Electronic equipment does not belong into domestic waste, but must be disposed in accordance with the applicable statutory provisions.

Description

This new WiFi otoscope is a portable, handheld, and practical ear inspection camera. The internal ear can be displayed clesly on your phone, and it will be a much helpful tool to analyze your ear health. The JPG images and AVI videos can be stored in the correspondent app.

— 01 —

This will be more convenient in your life, and if there is any irritation or horrible symptom in the ear, pls do not hesitate to go to doctor.

⚠ This appliance is not intended for medical use or personal inspection.

Safety

- ⚠ Keep the device dry, clean, free from oil, water, gas or other materials capable of causing corrosion.
- ⚠ This appliance is not intended for use by persons with limited physical, sensory or mental capabilities.
- ⚠ Keep bystanders, children away while operating the tool.
- ⚠ Do not overreach, keep proper footing and balance at all times.

Function

LED indicator

Long Press—Power ON/OFF

Short Press—Adjust brightness

Micro USB Charging port

— 02 —

How to Use WiFi Otoscope

Step 1:

Pls go to Apple Store or Google Play Store to download "Depstech " app for iOS or "Depstech-WiFi" for Android.

Download on the App Store

Get it on Google play

Step 2:

Long press the crystal plastic button, and LED lights and blue WiFi signal indication light will be solid on.

Step 3:

Go to your phone setting for WiFi list, and connect the WiFi named "WFE300_*****" via entering password "12345678". Pls kindly note that "*" is the symbol here, and there will be a series of random numbers and letters replacing them.

Step 4:

After connection, pls open the pre-installed app "Depstech" or "Depstech-WiFi". If the connection succeeds, the blue LED light will flash, and the image will be on.

— 03 —

Run the application

Take Photo Record Video Rotate 90° Captured Files Setting

Change the resolution of the camera:

1. Enter Settings;

2. Select the resolution;

3. Click on "Set Resolution".

640×480

1280×720

Charge the battery

⚠ The charging time will be almost 90 minutes with the green ligh on during charge. When the green light is off, the charge will be finished.

— 04 —

5V USB Charger (not included)

Technical Data

Description	Data	Data
Model	WFE300	WFE350
Camera Resolution	640*480 30fps 1280*720P 15fps	640*480 30fps 1280*720P 15fps
Camera Lens	100W/720P	100W/720P
Camera Voltage	3.3V	3.3V
Focal Length	15MM	15.3MM
Horizontal View Angle	70°	66°
Camera Power	150MAH*3.3V=0.5W	150MAH*3.3V=0.5W
Machine Power	260MAH*4.2V=1.1W	260MAH*4.2V=1.1W
Battery Capacity	360MAH	360MAH
Working Time	60-75Min	60-75Min
Charging Current	330MAH	330MAH

— 05 —

Safety Information

When operating your equipment:

⚠ WARNING:

This product is not intended for use in patient health care environments unless specifically designated.
This product is not designed for use in flammable or explosive environments.

⚠ WARNING:

Do not operate this equipment with any cover(s) (including bezels, panels, housings,)removed.

⚠ WARNING:

Do not use this equipment in a wet environments. Protect the equipment from liquid intrusion.

- Do not use damaged equipment, including exposed, frayed, or damaged power cords.
- Disconnect your device and all peripherals from any wall connection during and electrical (lightning) storm or when you will be away for extended periods.
- Do not attempt to service the equipment yourself.
- Contact us if your equipment does not operate normally.

Power Safety

When connecting your equipment to a power source:

- Check the voltage rating before you connect this equipment to an electrical outlet.
- Use only branded AC adapter approved for use with this device.
- Place the AC adapter and the this equipment in a ventilated area, such as desk top or on the floor, when you use it to charge the battery.
- The AC adapter and this equipment may become hot during charge operation.

— 06 —

Troubleshootin

• No image or defects, noise in the image

- * Check the Wi-Fi settings on your smartphone, make sure it has connected to the device.
- * Disconnect and reconnect the camera.
- * Close the app and open it again.
- * Check the battery level, recharge the battery.

• LED lights are dim or blink.

- * Battery Low.

• Forgot password.

- * Long press the crystal button for 3 seconds to reset. When blue WiFi indication light flashes three times, the reset will be okay.

— 07 —

NORTH AMERICA

E-mail:

support@idepstech.com(US)

support.ca@idepstech.com(CA)

EUROPE

E-mail:

support.uk@idepstech.com(UK)

support.de@idepstech.com(DE)

support.es@idepstech.com(ES)

support.fr@idepstech.com(FR)

support.it@idepstech.com(IT)

ASIA PACIFIC

E-mail:

support.jp@idepstech.com(JP)

MADE IN CHINA

www.idepstech.com

DEPSTECH

WFE300 / WFE350

P030000104

