

eutto

Professional Otoscope
User Manual

Quick start guide

1. Scan QR Code to download APP.



2. Remove the lid and the otoscope will be power on automatically.



3. Install the earpick firmly on the otoscope tip when you need to pick up your earwax .



4. Connect product WiFi: weiview-xxxx



5. Return to the App then click "start" to inspect ear.



eutto

Manufacturer:

SHENZHEN DEEPEA EXCELLENCE TECHNOLOGY CO., LTD

Address:

Fourth Layers, 4 Building, Jinhua hair Industrial Park, Sanlian community, Longhua District Longhua Street, Shenzhen, Guangdong, CHINA

E-mail:

US: support@eutto.com

UK: support.uk@eutto.com

JP: support.jp@eutto.com

Facebook:

@Eutto Official

Official Website:

www.eutto.com



FDA No.10061092



Made in China



6. The inside of the ear can be observed in real time by the App. And the transparent earpick won't block your view.



7. Screen lock is designed to avoid misoperation of children.



8. Low battery alert



When the battery is low, the red light flashes, please charge in time

Product size



Product parameters

Product name: Professional Otoscope

Product Model: WFE360

Image transfer rate: 25fps

Image transmitter: CMOS

Operating ambient temperature: 0 ~ 113°F

Battery type: Lithium battery

Battery life: approx 1 hour

Charging time: 1 hour

Input current: DC 5V 300ma

Probe diameter: 3.9mm

Optimnm focal length: 1.5-2cm

Gravity sensor: 3 Axis Gyro

Fault analysis and troubleshooting

Faults	Solution
No response on startup.	Try the correct boot method a few more time.
	Please charge with a matching charging cable for 1hour.
Camera doesn't light.	Shutdown and restart the otoscope.
APP crashing.	Uninstall the existing APP and rescan the QR code for installation.
Power indicator light flashes.	Short of power. Please use the matching charging cable for charging, and the charging time will last 1 hour.
The product failed to properly display real-time images after successfully connecting to APP.	Disconnect the phone from the otoscope WiFi, restart the otoscope, and reconnect the WiFi.
The WiFi connection is unstable and sometimes disconnected in seconds.	Restart the otoscope and reconnect the APP
If the above operation fails to solve the problem, please contact customer service, we will further provide you with a practical solution.	

Notes

1. Please observe the surroundings before use, and don't use the product while running to avoid impact.
2. Before use, be sure to install a protective pear-picking rod and fasten the head of the ear-picking rod to protect your ears.
3. This product is not suitable for children under 6 years old.
4. It is prohibited for children to use this product alone to avoid harm.
5. Please store the product in the normal temperature and avoid direct sunlight to avoid product damage.
6. The APP is not compatible with Samsung Galaxy S7.
7. When cleaning the lens, please wipe carefully with the professional alcohol cotton swab provided.
8. In order to ensure the best use experience, please keep the child in a seated position and operate the product horizontally when picking up the earwax for the child .
9. **ROHS compliance:** This product is in compliance with Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

FCC Requirement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Note:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.